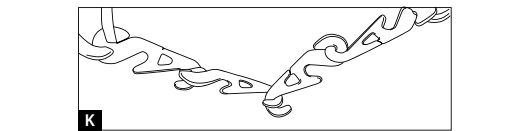
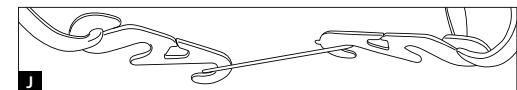
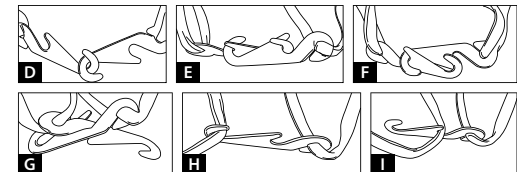
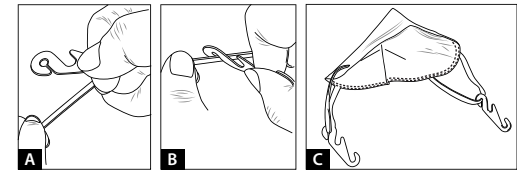
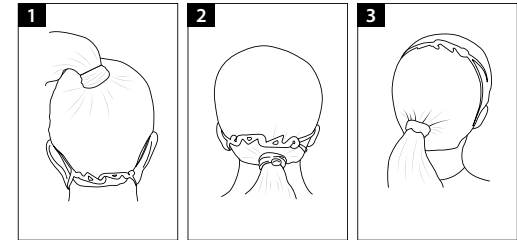
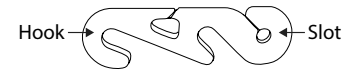
1. Unfold the respirator by opening the respirator front panels. The inner side is facing against your face.
2. Make flat on the metal nose strip by bending its center using thumb. Inspect the respirator for tears or damage.
3. Plastic clips are provided inside the packaging. Stretch the ear band (see figure A) then using the same hand to connect one plastic clip to an ear band by sliding the ear band into the slot at the end of the plastic clip (see figure B). Ensuring that the clip is connected to the ear-band firmly. Repeat the same to connect the other plastic clip to the other ear band (see figure C).
4. Use both hands to hold the two plastic clips. The metal nose strip is at the top. Position the respirator towards your face by pulling the ear bands onto your ear shells, ensuring that your mouth and nose are covered by the respirator.
5. Connect the two hooks at the end of both plastic clips together (see figure D). Ensuring that the ear band is flat and smooth against the back of your ear shell, without bending down your ear shell (see figure 1). To suit different users, the plastic clips can be connected in 3 ways - Neck (fig. 1), Occiput (fig. 2) for users with long hair, and Top (fig. 3).
6. There are several other hooks from the plastic clip for locking. If your head size is small, you must lock to further inwards (e.g. plastic clip's middle hook, see figure E or F) to increase the seal if you feel loosening otherwise there will be air leakage. Or choose other hooks to lock even more tightly (see figure G). Or even use single clip (see figure H and I). It is acceptable to use more than 2 pieces of plastic clips with the same connection method (see figure J and K). It is recommended that the maximum number of plastic clips is 4.
7. Notice that it is highly recommended to install permanently two plastic clips onto the two ear bands. For user with large head size, in this special case, it is acceptable not to use plastic clip if the user has passed the fit test and seal check (see point 11 and 12).
8. Use one hand to hold the metal nose strip and use another hand to pull down the bottom of the respirator to cover your mouth so that no pleat appears on the respirator.

No need to cover your chin fully, Most importantly is to fully cover your mouth and your nose.

9. The metal nose strip should span and cover the bridge of your nose. Use both hands to mold the metal nose strip to conform to the shape of your nose by pushing inward while moving your fingertips down along both side of the metal nose strip.
10. Lastly, re-adjust both ear bands to suit the best of your comfortability. Do not having your ear shells being pressed down by the ear bands otherwise there will be a possibility of your respirator being moved during your subsequent work.
11. Conduct a user seal check prior to each use. Place both hands completely to cover the respirator. Gently exhale and the respirator should bulge slightly. Then take a deep inhale and the respirator should collapse slightly. Otherwise, the respirator does not fit properly. Notice that it is unlikely that the requirements for proper seal will be achieved if facial hair passes under the face seal.
12. A more advanced seal check (Quantitative Fit Test per Europe CE EN149) is required for using in hazardous environment. As long as the Fit Test result is passed (fit factor > 100), it is flexible for the user to choose anyone of the donning methods said above to suit for different head sizes.
13. If disposable cap is worn, please ensure the disposable cap does not trap in between the face and the respirator to affect the sealing in between there.
14. If you feel leakage, re-adjust the fit of your respirator and check the seal again. Or start the donning step from the first step. If you can- not achieve a proper seal, do not enter a hazardous area.

Manufacturer:  
 Profit Royal Pharmaceutical Limited  
 Rm1211 & 1215, 12/F, Sunbeam Centre, 27 Shing Yip Street  
 Kwun Tong, Kowloon, Hong Kong  
 (852) 23878926 (852) 23878901 info@ptrl.com.hk www.ptrl.com.hk PROFIT ROYAL

**innocare swiss**

Importer/Distributor:  
 Innocare Swiss Health GmbH  
 Blegistrasse 13, 6340 Baar, Switzerland  
 T: +41 41 520 76 60 | E: info@innocareswiss.com | http://www.innocareswiss.ch

Discover more at:

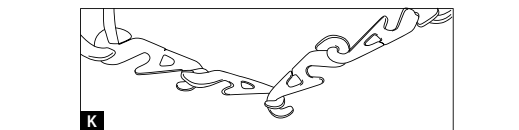
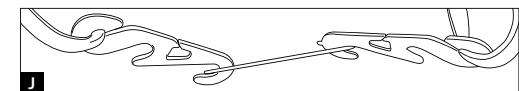
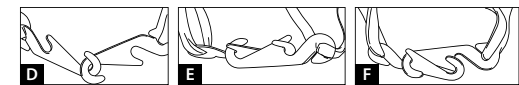
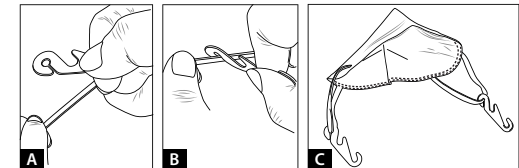
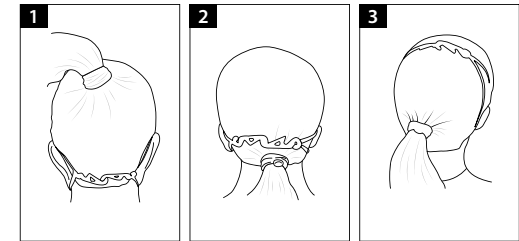
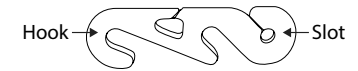
*Important Notice: If proper fit test result cannot be achieved, do not enter contaminated areas. To the extent permitted by law, Profit Royal Pharmaceutical Ltd. shall not be liable for any loss or damage including any loss of business, loss of profits, or for any indirect, special, incidental or consequential loss or damage arising from reliance upon any information herein provided by Profit Royal Pharmaceutical Ltd. Nothing in this Instructions for Use (IFU) will be deemed to exclude or restrict Profit Royal Pharmaceutical Ltd. liability for any personal injury arising from its negligence.*

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Manufacturer:

Profit Royal Pharmaceutical Limited  
Rm1211 & 1215, 12/F, Sunbeam Centre, 27 Shing Yip Street  
Kwun Tong, Kowloon, Hong Kong  
(852) 23878926 (852) 23878901 info@prtl.com.hk www.prtl.com.hk

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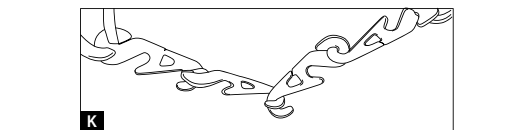
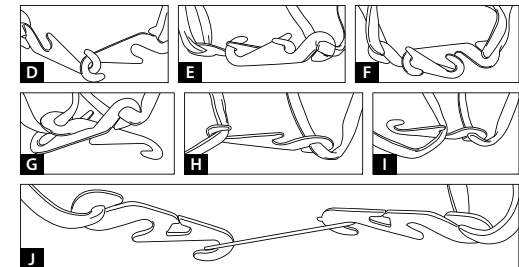
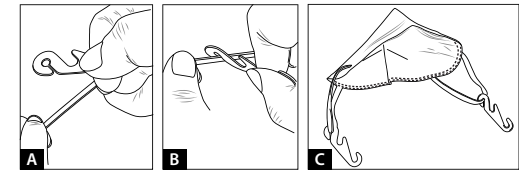
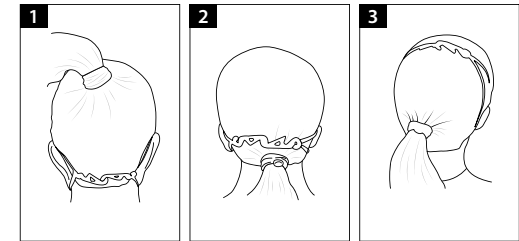
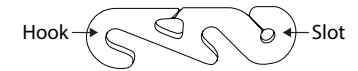
Manufacturer:

Profit Royal Pharmaceutical Limited  
Rm 1211 & 1215, 12/F, Sunbeam Centre, 27 Shing Yip Street  
Kwun Tong, Kowloon, Hong Kong  
(852) 23878926 (852) 23878901 info@ptrl.com.hk www.ptrl.com.hk

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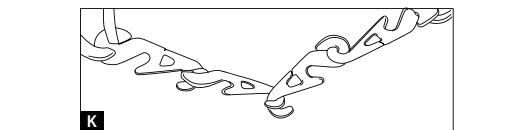
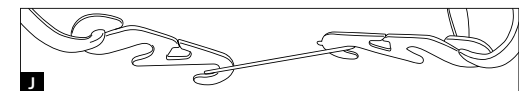
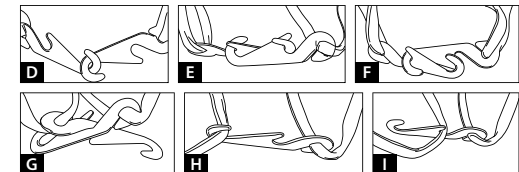
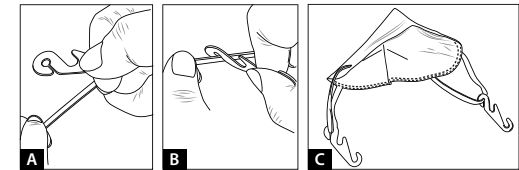
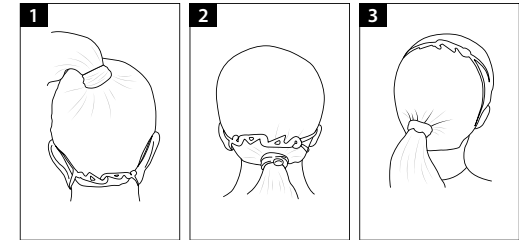
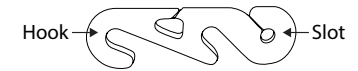
Manufacturer:

Profit Royal Pharmaceutical Limited  
 Rm1211 & 1215, 12/F, Sunbeam Centre, 27 Shing Yip Street  
 Kwun Tong, Kowloon, Hong Kong  
 (852) 23878926 (852) 23878901 info@ptrl.com.hk www.ptrl.com.hk

PROFIT ROYAL

**innocare swiss**

Importer/Distributor:

Innocare Swiss Health GmbH  
 Blegistrasse 13, 6340 Baar, Switzerland  
 T: +41 41 520 76 60 | E: info@innocareswiss.com | http://www.innocareswiss.ch

Discover more at: